# RollerMate®

## User manual

Rollermate G2 Basic

Nederlands

English

Deutsch

For more info and support visit [www.rollermate.se](http://www.rollermate.se)

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**Warning!** Risk of injury cases. The product is designed for users with sufficient balance and ability to anticipate environmental factors and resist unexpected movements. If you are unsure whether the device is suitable for you, ask a knowledgeable person for advice.

**Warning!** risk of slip injury. The product should not be used in slippery conditions.

**Warning!** Be careful when folding not to pinch hand or fingers.

Please note! Risk of musculoskeletal disorders by prolonged static use.

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## Height adjustment

Please note! It is important that the correct height is used. If unsure, please consult an expert. Make sure the pin is fully inserted!

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## Folding

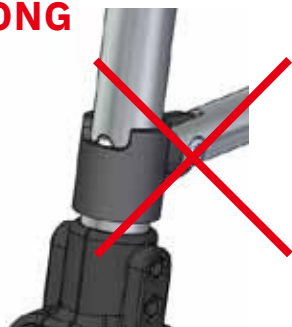
Push button to fold. When in use, the sliding part shall be in its lowest position.

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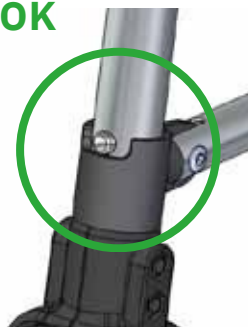
## Care and Maintenance

Periodically check that no cracks or other visible damage exists and that no screws are loose. Wipe of dirt with a damp cloth and mild detergent. Wipe it dry if it gets wet.

## Warranty

We give two years warranty on your RollerMate if being used normally. This does not apply to wear items. We reserve the right to make product changes without prior notice.

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